

## ECG DIAGNOSTIC ELECTRODE


### MR CONDITIONAL



FOR USE WITH:


Product No	Reference No	Product No	Reference No
6500050V	A10031-50RT	6502690V	Z05MTRT-5
6500052V	A10031-60RT	6502691V	Z05MTRT-50
6500046V	A10025-60	6502696V	Z05SGRT-30
6502952V	A10032RT-5NB	6502697V	Z05SGRT-50
6500074V	A10083-3	6502724V	Z35RT-3
6500075V	A10083-5	6502725V	Z35RT-5
6502686V	Z05MCRT-50	6502726V	Z35RT-50

#### DURATION OF USE

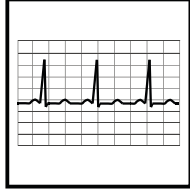


● < 72 hours


### INSTRUCTIONS FOR USE

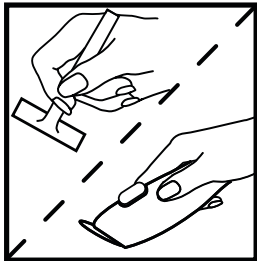


• Shave/Clip

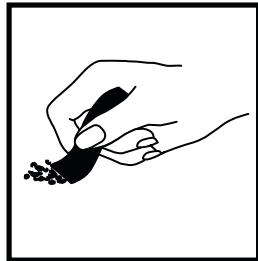


• Prep

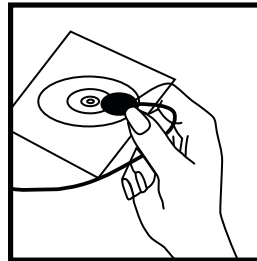

• Connect

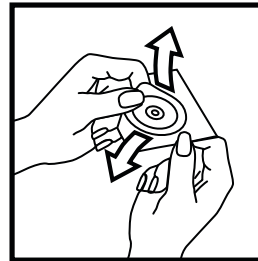

• Peel

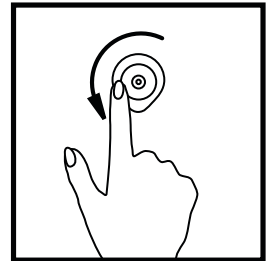

• Apply

### CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

### PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML175 (04/2021)